

Tracheal Tube For Single Use


(一次性氣管導管的使用)

Propose Using !

To operate in coordination using for the
Clinical Three Channel Valve

Caution and instruction manual:

1. Store in clean, dry condition away from light and avoid freezing and excessive heat;
2. Seat 15mm connector firmly into tracheal tube prior to use;
3. Avoid contact with laser beams or electrosurgery electrodes;
4. To tear open the parcel bagging and take out the Tracheal Tube to join the contact;
5. Using an injector to face the gasbag to fill an little gas to check if puncture of the gasbag;
6. To pump out gas of the gasbag;
7. Start the normal operation to intubate procedure;

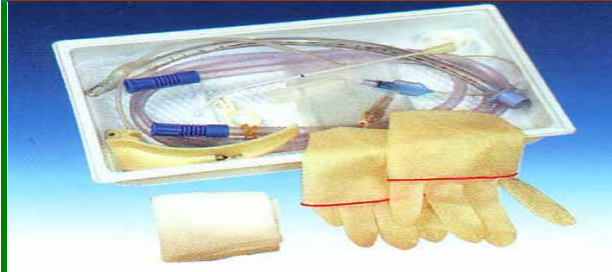

HKD \$30.00

(single tube type)



Children type: 2.5#, 3.5#, 4.5#, 5.5#, 6#;
Adult type: 6.5#, 7.5#, 8#, 8.5#, 9#, 9.5#, 10#;

HKD \$50.00 (repeatedly use)
Clinical Three Channel Valve



Common Tracheal Tube Package
For Single Use

Even tube Tracheal Tube for Single use


high-priced !

Standards Diameter

Children type: 2.5#, 3.5#, 4.5#, 5.5#, 6#;
Adult type: 6.5#, 7.5#, 8#, 8.5#, 9#, 9.5#, 10#;



建議使用 最新的改良型

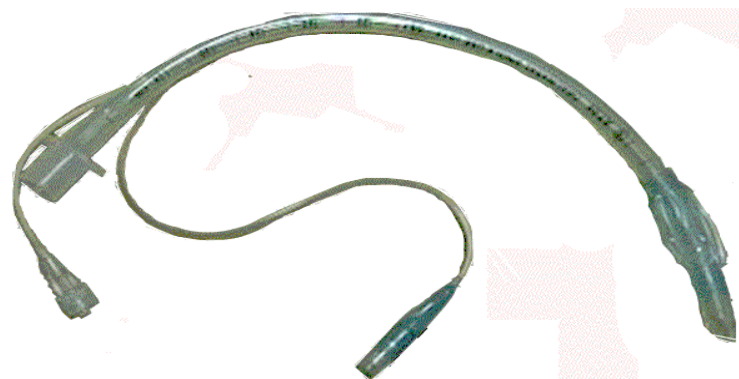

輸液口

吹氣閥

由此插入支器管